

BloodSTOP®

CONTROLS BLEEDING INSTANTLY

Instructions

BloodSTOP® is a water soluble gauze which stops bleeding in a matter of seconds. Upon contact with blood, it speeds coagulations and expands into a transparent gel that applies pressure on wound as it seals it. This transparent seal aids in wound monitoring. It is ideal for bleeding disorders.

HOW TO USE BloodSTOP® for bleeding wounds, cuts, scrapes or razor nicks:

1. Cut a piece of BloodSTOP® slightly larger than the wound area.
2. Dip the BloodSTOP® in water for just one second, and then apply over the wound.
3. If the wound is moist or blood is flowing, skip step 2 and apply BloodSTOP® directly to the wound.
4. Keep BloodSTOP® moist with water or petroleum jelly.
5. After bleeding completely stops, BloodSTOP® can be washed away with water.



HOW TO USE BloodSTOP® for nosebleeds:

1. Cut BloodSTOP® into a small strip. Use dry tweezers to pack the BloodSTOP® strip into the nostril (nose). After bleeding stops completely, wait at least one hour. If necessary, use water to rinse it off.
2. Compress the bleeding nostril if necessary.

HOW TO USE BloodSTOP® for Dialysis and Venipuncture:

1. Dip the BloodSTOP® in water for one or two seconds until it is completely soaked.
2. Apply one BloodSTOP® gauze over the skin puncture.
3. Keep BloodSTOP® moist with water or petroleum jelly.
4. After the bleeding completely stops, BloodSTOP® can be washed away with water.

